Note: Document originally drafted in the English language.

Product Description

The B-150H housing is designed for Varian rotating anode inserts having 102mm (4 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:

Housing 19.5 kg (43 lbs)
with Trunnion Rings 22.9 kg (50.5 lbs)
with Trunnion Rings and Fan Assembly .. 24.1 kg (53.2 lbs)

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Cable de la Bovina 0.9 m (3 ft)

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Produktbeschreibung

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
Nominal X-ray Tube Voltage	150kV
Maximum Cathode to Ground	75 kV
Maximum Anode to Ground	75 kV
Grid to Cathode (If applicable)	-4 kV
Grid Control Voltages	
Typical Bias Voltage for Cutoff at 150 kV	-3600 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-Ray Tube Assembly Heat Content ..	1,111 kJ (1,500 kHU)
Maximum Continuous Heat Dissipation (max. housing temperature 78°C)	
without air circulator	200 W (270 HU/sec)
with air circulator	400 W (540 HU/sec)
X-Ray Tube Assembly	
Permanent Filtration	0.7mm Al/75 kV IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	
Humidity	-20°C to +75°C
Atmospheric Pressure Range	+10% to +90%
	70 kPa to 106 kPa
X-Ray Tube Assembly.(Complies to)	IEC 60601-2-28

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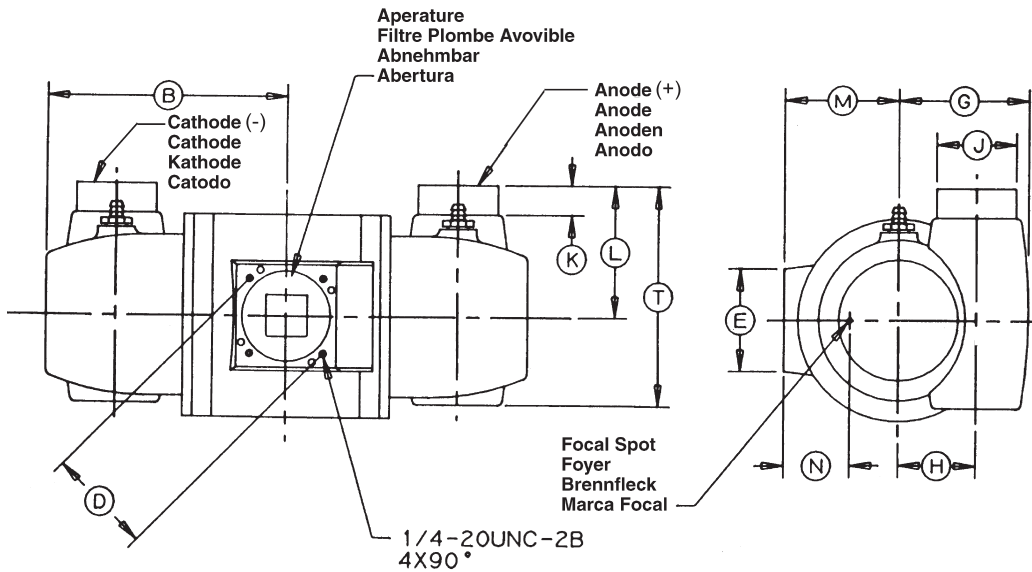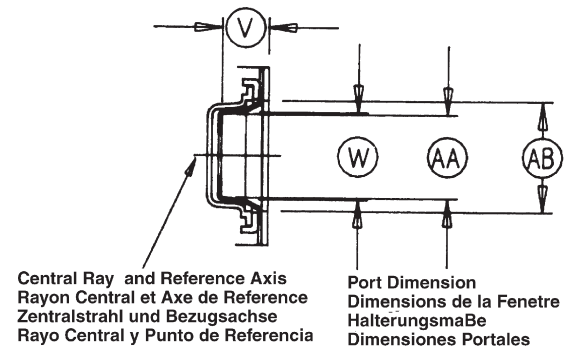
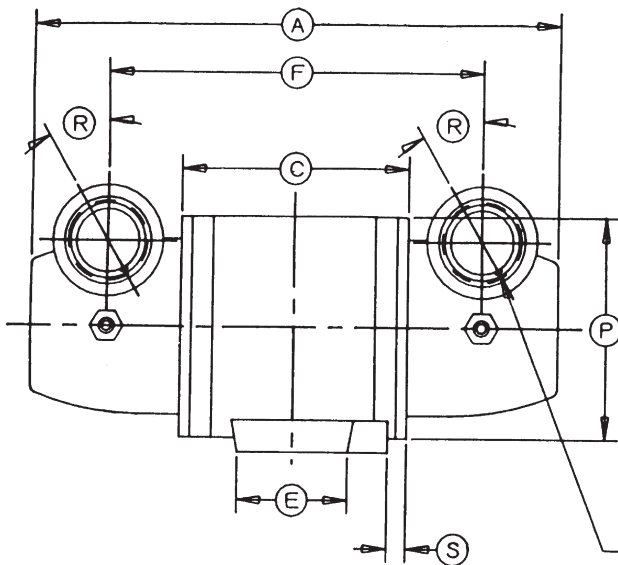
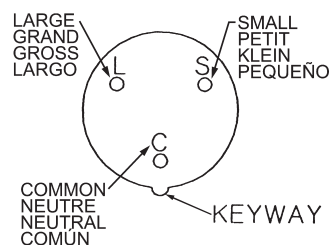
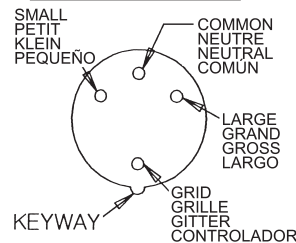
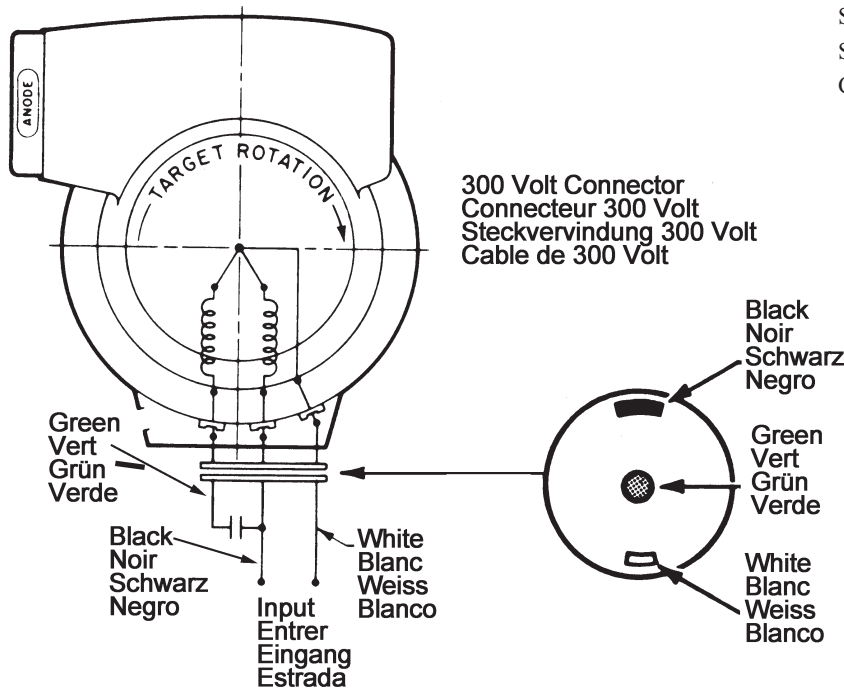
TO ENSURE PROPER TARGET ROTATION, EACH STATOR CORD IS EQUIPPED WITH A SPECIAL TWIST-LOCKING CONNECTOR AND A MATING CONNECTOR.

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Stator Power:

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varian stator types are rated for regular speed and high speed starters. Time to full speed of 4 inch (102mm) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 500 volts rms.

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